GC Acrylic Primer

Light-curing primer for bonding composites to acrylics

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

Increases the adhesiveness of light-curing composites to conventional acrylic resins used in dental laboratory procedures (e.g. modification of denture teeth / denture base resins).

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methyl methacrylate monomer, poly methyl meth- acrylate polymer or acetone.

COMPOSITION

Acetone, methylacrylate, dimethacrylate, multifunctional monomer, initiator, stabilizer

DIRECTIONS FOR USE

- Preparation of adhesion area.
 - Roughen the surface by sandblasting with aluminum oxide (50µm at 2bar) or by roughening by tungsten burs. Clean the surface by steam cleaner or clean with oil free air. The prepared surface should not be touched anymore.
 - Applying GC Acrylic Primer. Disperse a few drops of GC Acrylic Primer into a dispensing dish. Wet the adhesion area with GC Acrylic Primer by using a brush and light-cure the area to polymerize the material.
- Irradiation time and curing unit

Curing Unit	LABOLIGHT DUO	STEPLIGHT SL-I	LABOLIGHT LV-III, LV-II
	Step-mode	Full-mode	Pre-cure
			Final cure
GC A. Pr.	20sec	3min	20sec, 3min

- NOTE:**
- Light-cure completely for an effective bonding strength.
 - Use a protective light shield or similar protective eye wear during light-curing.
 - Use fresh material and avoid leaving the material dispensed for a long period of time. Its high volatilization may cause insufficient curing or may decrease the bonding strength if it has been left too long on a dispensing dish.
 - Remove excess material to avoid any cause of decrease of the bonding strength.

- Layering of composite resin materials. Before layering make sure the to be layered surface is free of any impurities. Place light-curing composite resin materials on the adhesion area and polymerize them in accordance with the manufacturer's instruction for use (e.g. GRADIA PLUS (GCS)).

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

PACKAGES

GC Acrylic Primer, Bottle 6ml.

CAUTION

- In rare cases, the product may cause sensitivity in some people. If such reaction occurs, discontinue the use of the product and refer to a physician.
- The device cannot be used by other than a dental professional.
- The device cannot be used for any indication other than in accordance with the manufacturer's instruction for use.
- When cutting and polishing the prosthesis, wear a safety goggle, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Don't reuse a brush with other products.
- When carrying out the work, to provide sufficient ventilation to prevent headache etc. If he is inhaled by human, move to fresh location.
- Avoid inhalation or ingestion of material.
- Do not use wear naked face. Keep away from sources of ignition. Keep away from direct sunlight.
- If not in use for a prolonged period of time, store in refrigerator, otherwise store at room temperature. GC Acrylic Primer is to be used at room temperature.
- GC Acrylic Primer is to be used in a well-ventilated place. Apply to adhesion area and close the bottle quickly after application.
- Before closing the liquid bottle we advise to clean the nozzle. Always vertically close the bottle.
- In case of contact with eyes, rinse immediately with water and seek medical attention.
- In case of contact with oral tissue or skin, immediately remove with a sponge or cotton soaked in alcohol. Rinse immediately with water.
- In case of surface spills, immediately wipe off with a dry cloth.
- Do not mix with other similar products.
- Dispose all waste in accordance with local regulations.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referred in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact us at Regulatory@gccg.dental

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gccg.dental

In this way you will contribute to improve the safety of this product.

Last revised: 10/2025

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact us at Regulatory@gccg.dental

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gccg.dental

In this way you will contribute to improve the safety of this product.

Last revised: 10/2025

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact us at Regulatory@gccg.dental

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gccg.dental

In this way you will contribute to improve the safety of this product.

Last revised: 10/2025

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact us at Regulatory@gccg.dental

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gccg.dental

In this way you will contribute to improve the safety of this product.

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